MEDIDRAPE, MEDIDRESS

Non sterile: drapes, sets of drapes, surgical sets, surgical gowns, surgical clothing, sets of surgical clothing

Unsterile: Abdecktücher, Abdecktücher-Sets, OP-Sets, OP-Mäntel, OP-Bekleidung, OP-Bekleidung in Sets

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EN

PRODUCT DESCRIPTION:

Medidrape and Medidress products are reusable medical devices, intended for the operating theater after prior sterilisation.* The drapes and clothing meet the requirements of the standard EN 13795.

PRODUCT USE AND CONSERVATION:

1/ The products can undergo max. 100 use cycles* including all the following stages: washing, drying, sterilisation*, applying in the surgical suite. The in-saw label sewn on the drape or the gown enables to mark every next cycle of using the product*.

2/ Product maintenance should be carried out according to the guidelines placed on the sewn-in label in form of graphic symbols in accordance with EN ISO 3758. The combination of symbols depends on how the product is to be maintained. The drying of surgical gowns and surgical clothing can be done with the use of the Finisher tunnel, where the recommended temperature should not exceed 130°C.

 Maximum washing temperature 70°C. Wash separately. Wash only in washer extractors or continuous batch washers completed.

 Tumble dry possible at normal temperature

 Drip dry

 Do not bleach

 Do not dry clean

 Ironing, maximum 150°C

 Do not iron

- the above symbols are placed on the sewn-in label attached to the product - their choice depends on the conservation method.

3/ Sterilise by hot steam in a validated cycle (134°C, 4 - 10 minutes), after repacking into a sterilisation packaging. The repacking must be done by qualified staff meeting GMP requirements concerning personal hygiene, in controlled environment meeting the requirements of air microbiological cleanliness max. 300cfu/m³, and air dust cleanliness for class 8 according to the actual standard EN ISO 14644-1.

4/ Recommendations:

- washing the product is required before first use
- for washing use available washing agents for washing and disinfecting at the temperature up to 70°C;
- do not use antistatic, antibacterial or softening agents;
- pH value of the washing bath cannot exceed 10;
- the load during washing and drying may not exceed 70% of the nominal capacity of the device;
- do not use sharp instruments for fixing the drapes for they may damage the fabric.

5/ Caution!

The manufacturer does not guarantee the preservation of product properties in the event of non-compliance with the provided washing and sterilisation regulations. If the product is damaged, the product should be withdrawn from use.

* not applicable to surgical clothing

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