

Codosil ADHESIVE is a soft silicone dressing used to reduce the size of hypertrophic scars and keloids, and improves their colour and elasticity so it becomes less visible.

1



Take out the dressing from the packaging and adjust it to the shape of the scar with scissors. The dressing should be about 1 cm wider at the edges than the size of the scar. There is no need to immediately use up the entire dressing, e.g. if we treat a much smaller scar. It can be used piece by piece.

2



Peel off the white protective film that protects the adhesive layer of the dressing.

3



Stick the dressing onto the scar surface. Adhesive properties of the dressing allow for its repeated sticking onto the skin surface. Before every next use the dressing needs to be washed.

4



In order to prevent accidental removal of the dressing from moving body parts it is recommended to use an elastic dressing mesh Codofix®. The mesh is available in 8 different widths, which enables precise selection of the product to the perimeter of the dressed body part.