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INSTRUCTIONS FOR USE
CODOFIX® PLUS Elastic dressing mesh

- CHARACTERISTICS**
Codofix® PLUS elastic dressing meshes are used for supporting dressings in lieu of a traditional bandage. The use of **Codofix® PLUS** makes the wound dressing process easier both for the patient and the medical staff.
- INDICATIONS**
Codofix® PLUS meshes are used for:
 - fixing a dressing on hard-to-reach body parts such as pelvis and crotch,
 - intraoperative wounds of abdomen, thorax and head,
 - supporting dressings on scald wounds,
 - pediatric surgery.
- PRODUCT RANGE**
 Tricomed S.A. manufactures **Codofix® PLUS** meshes in form of a sleeve of the length 25 m (stretched) in 5 sizes:

Size	Width*(cm)	Use
P1	1,0	finger
P2	2,5 – 3,0	palm, forearm, foot, elbow
P3	3,5 – 4,0	arm, shank, knee, foot
P4	5,5 – 6,0	thigh, head
P5	over 10,0	thorax, abdomen, hips

- * relaxed
- CAUTION**
 - The mesh size should be chosen according to the patient's individual silhouette.
 - The product must not be used after the expiry date.
 - The product is non sterile.
 - Codofix® PLUS** meshes must not be used under plaster casts.
 - Do not use in people with hypersensitivity to the product components: polyamide and polyurethane
 - SYMBOLS ON THE PACKAGING**

	batch code		Medical Device
	date of manufacture		
	use by		
	do not reuse		
	do not use if package is damaged		
	keep away from sunlight		
	keep dry		
	consult instructions for use		
	manufacturer		

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