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INSTRUCTION FOR USE
CODOFIX® Elastic dressing mesh

- CHARACTERISTICS**
Codofix® elastic dressing meshes are used for supporting dressings in lieu of a traditional bandage. The use of **Codofix®** makes the wound dressing process easier both for the patient and the medical staff.
- INDICATIONS**
Codofix® meshes are used for:
 - fixing a dressing on hard-to-reach body parts such as pelvis and crotch,
 - intraoperative wounds of abdomen, thorax and head,
 - supporting dressings on scald wounds,
 - pediatric surgery.
- PRODUCT RANGE**
Tricomed S.A. manufactures **Codofix®** meshes in form of a sleeve of the length 1 and 10 m (relaxed) in 8 sizes:

Size	Width*(cm)	Use
1	max. 1,5	finger
2	1,5 – 2,5	palm, finger
3	2,5 – 3,5	hand, foot
4	3,5 – 4,5	shank, knee, arm, foot
6	5,0 – 6,5	head, arm, shank, knee
8	7,0 – 9,5	thigh, head, hips
10	10,0 – 13,5	hips, abdomen
14	min. 14,0	thorax, abdomen, hips

* relaxed

- CAUTION**
 1. The mesh size should be chosen according to the patient's individual silhouette.
 2. The product must not be used after the expiry date.
 3. The product is non sterile.
 4. **Codofix®** meshes must not be used under plaster casts.

SYMBOLS ON THE PACKAGING

- batch code
- date of manufacture
- use by
- do not reuse
- do not use if package is damaged
- keep away from sunlight
- keep dry
- attention, see instructions for use
- manufacturer
- Medical Device
- MANUFACTURER / HERSTELLER / ИЗГОТОВИТЕЛЬ / WYTWÓRCA: TRICOMED S.A. Swiętojarska 5/9, 93-493 Łódź, Poland tel. +48 42 689 65 20, fax. +48 42 684 68 74 tricomed.com

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