

In order to the secure places at risk of abrasions or moisture access, you need to choose the right size of the Medisorb F dressing that will cover the entire affected area of the skin. Before applying the dressing put on gloves and well clean and dry the skin. The next step is to apply the dressing.

**1**

Open the packaging and take out the dressing.

**2**

Remove the backing paper (white) from the central bottom part of the dressing and holding by the paper edges apply the dressing on the skin.

**3**

Place the dressing on the skin.

4



Remove the remaining backing paper (white) of the edges and stick the rest of the dressing. Gently press the edges of the dressing.

5



Remove the top plastic cover by pulling the tabs in the direction indicated by the green arrows.

6

