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Neurocompress

EN
NEUROCOMPRESS
NEUROSURGICAL SWAB STERILE

DESCRIPTION
 Neurocompress is an absorbent sterile medical material used for absorbing cerebrospinal fluids and blood during neurosurgical procedures. The neurosurgical swabs are fixed on a cardboard dispenser used for stable placement of the product in the packaging. Neurocompress is composed of:

- absorbent element,
- X-ray detectable thread.

ACTION
 Neurocompress absorbs body fluids during neurosurgical procedures. The X-ray detectable thread firmly attached to the absorbent element enables to easily detect the product in the OP-field.

INSTRUCTIONS FOR USE
 1. Open the packaging according to the marking on the label.
 2. Take off the neurosurgical swabs from the cardboard dispenser.
 3. Place in the OP-field (specified by the specialist) for 60 minutes.
 4. Remove from the OP-field, remembering that the thread is used for identification only.

REMARKS
 Do not use the X-ray detectable thread to pull the swab out of the wound. Use the products only in the form and size provided. Neurocompress is the only product acceptable to get into contact with the cerebrospinal fluid. The reuse of the product is unacceptable because of the high possibility to infect the patient.

Neurocompress is available in the following sizes:

10 mm x 10 mm	25 mm x 25 mm
10 mm x 50 mm	25 mm x 75 mm
10 mm x 100 mm	25 mm x 90 mm
15 mm x 15 mm	25 mm x 100 mm
15 mm x 100 mm	30 mm x 30 mm
20 mm x 40 mm	30 mm x 50 mm
20 mm x 50 mm	30 mm x 75 mm
20 mm x 60 mm	30 mm x 90 mm
20 mm x 100 mm	50 mm x 90 mm

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